


# SPASY™

## Bone Cement Dispensing Kit


### Indication

- VCF(Vertebral compression fracture)due to Osteoporosis
- Osteolytic fracture
- Metastatic bone fractures



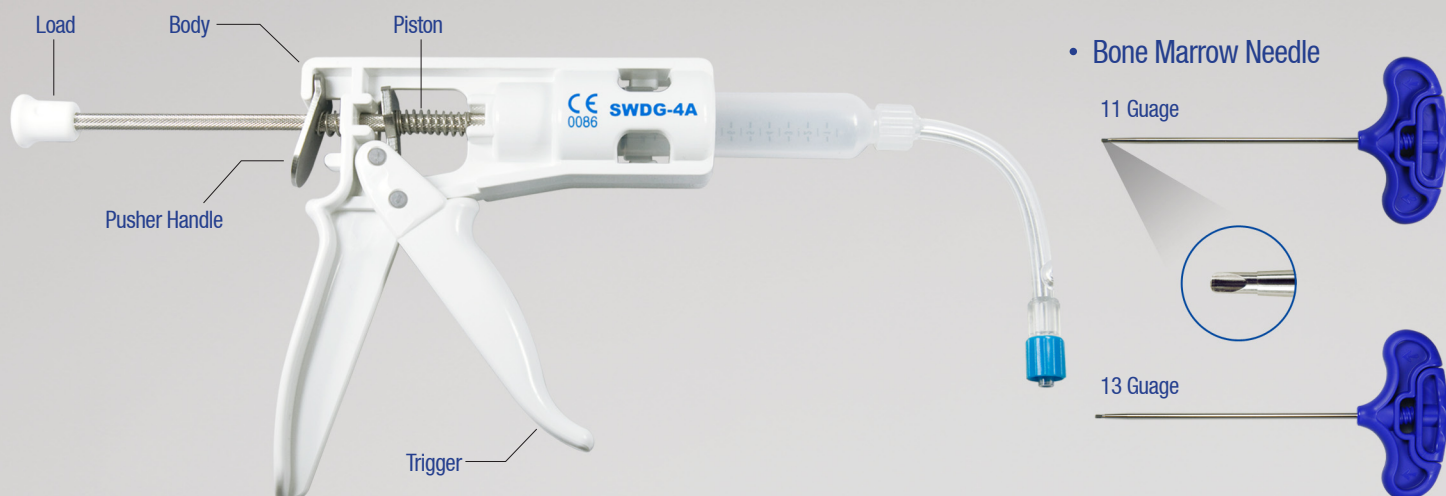


### Step-1

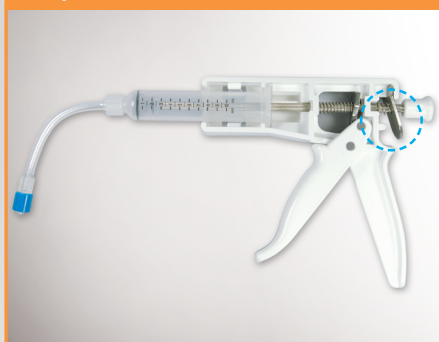


Push the pusher plate and pull out the load to fix the piston in the middle of the body.

Push the Pusher Handle

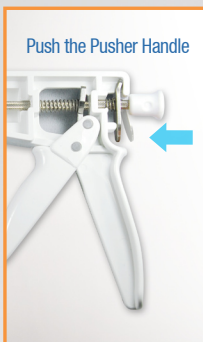

Pull out the Load

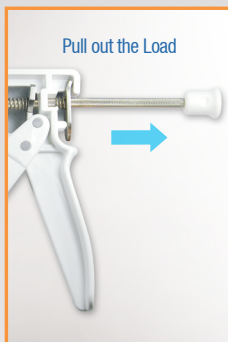

### Step-2



Pour the cement carefully into the syringe leaning to the left side as presented above

### Step-3



Turn the syringe to the right till the "Lock sound" is audible. Then, pull the trigger to remove the air inside of the syringe.

### Step-4



Assemble the hose to the Bone Marrow Needle. Pull the trigger with the handle slowly to press the piston in the syringe to push the bone cement.

### Ordering Information

| Part No    | Description                | EA/Kit     |
|------------|----------------------------|------------|
| SWDG-4A    | Bone Cement Dispensing Kit | 1EA        |
| SWVP-11A   | Bone Marrow Needle         | 2EA        |
| SWVP-13A   |                            |            |
| Article No | Gauge                      | Length(mm) |
| SWVP-11A   | 11                         | 125        |
| SWVP-13A   | 13                         | 125        |