



Efficiency
High quality
Ergonomic Design

MIX MAX


Bone Cement Mixing System

MIX MAX is a device for high quality cement mixing.
By using this, the efficiency of surgery can be increased.



Feature

- **Ergonomic Design** Ergonomic design allows easy mixing.
- **Efficiency** Waste of cement can be minimized by using spatula.
- **High quality** The cement density can be increased by mixing in vacuum.

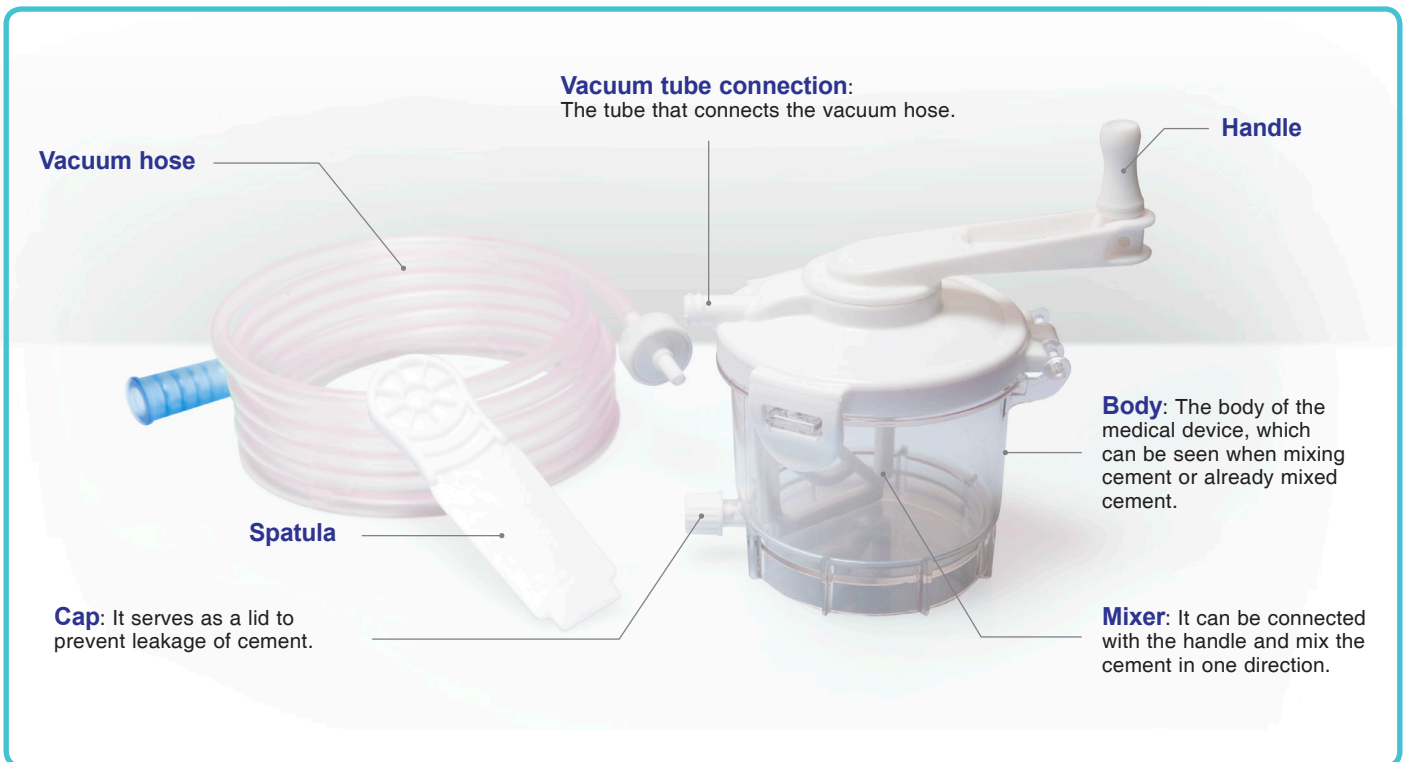

How to operate

- 1 Open the package.
- 2 Open the lid of the cement mixer.
- 3 Connect the filter connector of the vacuum hose to the vacuum tube connection of the cement mixer and connect the vacuum connector of the vacuum hose to the vacuum
- 4 Pour medical bone cement into container of cement mixer.
- 5 Close the lid of the open cement mixer.
- 6 Activate the vacuum machine and mix the cement.
- 7 When mixing is complete, stop the vacuum and disconnect the vacuum hose.
- 8 Open the cement mixer.
- 9 Remove the mixed cement with spatula.

Ordering Information

Article Name	Component Article Name	Device Name	Q'ty(ea)
SWMX-CMS-01	SWMX-BC-01	Cement mixer	1EA
	SWMX-VT-01	Vacuum hose	1EA
	SWMX-SP-01	Spatula	1EA

Caution

Disposable products must be treated with medical waste after use.